Save the gonads

www.savethegonads.com

Save the Gonads, Ltd.
5710 Sheits Rd.
Cincinnati, OH 45252 USA
info@savethegonads.com
Registered Design No.: 39219-0001
Manufactured in the USA

Thank you for giving
a Small Life a Bigger Future!

HOW TO USE AN STG

(Save the Gonads)

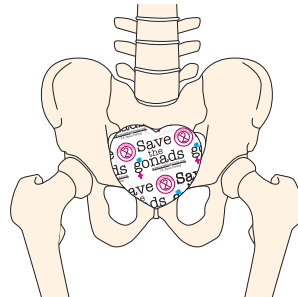
Every admission to your unit should have the appropriate size STG at their bedside.

The STG can be used for male or female babies. With the flip of the heart both sexes are protected. The heart remains upright for the female, to cover the ovaries. For the male, the heart is flipped downward to cover the testicles.


The STG is lead encased in a soft satin covering and can be easily cleansed with an alcohol wipe. For infection control reasons the STG is meant for individual patient use only. The STG may be reused repeatedly for a baby, however, the STG should not be shared with other babies.

The STG contains lead and needs to be disposed of properly. Remind parents that if they do keep this as a souvenir, that it must be kept out of the reach of children.


When used properly the STG significantly reduces radiation exposure to the gonads. It is a tool only to be used by qualified medical personnel.


Positioning for Girl

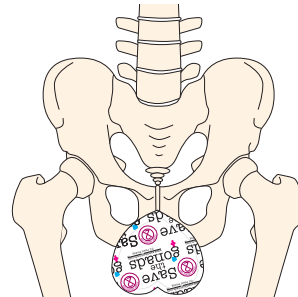Positioning for Boy

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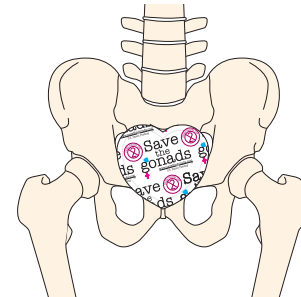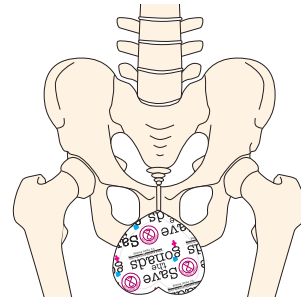"Save the Gonads" is not responsible for misalignment of the STG, causing radiation exposure to the gonads.

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