



The Stopper by CHS - INSTRUCTIONS FOR USE

- Place surgical tray on top of desired sterile wrap.
- Lift each bottom corner of surgical tray and slide **The Stopper Foot and Corner Protector** under each foot and/or corner of tray.
- Repeat the above, only invert **The Stopper Foot and Corner Protector** to the top corners of the tray.
- If desired for maximum protection, place **The Edge Protector** on each side of the tray
- Once the protectors have been applied to all corners and edges, continue to wrap the surgical tray as per hospital/clinic/surgery center protocol.
- Once surgical tray is used/opened, return corner guards to Decontamination area.



The Stopper - Cleaning Instructions

Point of Use:

Remove soil with disposable cloth/paper wipe.

Cleaning:

Equipment:

Detergent, brush, running water

Note: Use only neutral (6.0-7.5) pH solutions, mildly alkaline and free of sodium carbonate to avoid damaging product. DO NOT USE scouring pads or abrasive cleaners.

Gross decontamination:

1. Remove all visible soil and contaminants using a soft bristle brush. The entire product should be immersed while cleaning to aid in the removal of contaminants and to reduce splashing of detergent on personnel.
2. Rinse thoroughly with clean water to remove all detergent. See rinsing instructions on the detergent label.

Automated Washer:

The **Stopper** cleaning process is reflected in the cycle detailed below:

Pre-Wash 1 & 2	
Re-circulation Time	2 minutes
Water Temperature	Cold Tap Water
Enzyme Wash:	
Soaking Time	(1 oz/gal)
Water Temperature	4 minutes
Water Temperature	Hot Tap Water
Wash 1:	
Re-circulation Time	15 minutes
Water Temperature	Heated at 65.5°C
Rinse 1 & 2	
Re-circulation Time	5 minutes
Water Temperature	Hot Tap Water

Note: After completion of an automatic washer cycle, visually inspect the **The Stopper** for any remaining soil. **All visible soil must be removed by hand cleaning, brushing and additional automatic cycles prior to sending to sterilization.**