

# Disposable Covers for Swings & Rockers with 3-Point Harness

### **CoverMe Specifications**

Medical grade, breathable, waterproof fabric. 100% polypropylene cover. Latex-free. Fabric meets AAMI PB70 Level 3 testing. **Single-patient use**, disposable. For hospital and home use. Do not leave baby unattended in swings or rockers. Manufactured in the USA.

#### Instructions For Use

- For use with three point harness infant swings like the mamaRoo®.
- Swing seat cover must be used at all times the CoverMe goes over the seat supplied with the swing. • Single patient use.
- DO NOT share covers between babies due to the risk of infection and cross contamination.
- May be spot cleaned with hospital spray cleaners and sani-wipes while in use with the same patient.
- •Cover may be used for baby's length of stay if material is not soiled, damaged, cut or torn. Replace if any of those conditions occur.
- Cover may be stored in a "patient belonging" bag when not in use.
- •Use swing harness at all times. Do not leave baby unattended in swing/rocker.

## Do I still need to wash the cloth seat that comes with the mamaRoo?

You should follow your hospital protocols. In general, if the cloth seat is always covered by the CoverMe and the CoverMe is not damaged, cut or torn you would not need to machine wash the fabric seat. Wipe down the fabric seat and the entire swing between patient use. If the CoverMe is excessively soiled or if the soiling gets on the fabric seat, the fabric seat should be removed and machine washed according to swing manufacturer's instructions. Clean the mamaRoo seat per hospital infection control standards between patient use.

# How to put on the CoverMe:

1. Pull crotch strap through crotch extension.



4. Fold top flap down, Velcro in place.



2. Pull side straps through strap extensions, lock into crotch straps.



5. Tuck extensions around & under baby to secure.



3. Tuck flap of crotch extension in.



Watch the video https://youtu.be/21-6MntpIUo